checkToP®

1000 mUI/mL

A self-test to check the outcome of First Trimester Termination of Pregnancy (ToP) or miscarriage

For the measurement of hCG above 1000mIU/mL

INTRODUCTION

checkToP® test is not a classical pregnancy test. checkToP® is a low sensitive urinary rapid test for the determination of the hCG (human Chorionic Gonadotropin) level in urine for home use, to determine the outcome after a previous Termination of intrauterine Pregnancy (ToP) of first trimester and miscarriage.

Your Health Care Professional will explain you if and how to use this test.


CONTENTS

Ref. 3066-1000AI: Each pack contains one (1) instruction for use and one (1) checkToP® test packaged in a hermetically sealed aluminium pouch containing one desiccant sachet.

Ref. 3067-1000AI: Each pack contains twenty (20) instructions for use and twenty (20) checkToP® tests packaged in a hermetically sealed aluminium pouch containing one desiccant sachet.

PRINCIPLE OF THE TEST

checkToP® is a rapid qualitative test for the presence of hCG in urine. In 5 to 10 minutes, it will detect an hCG level of 1000 mIU/mL or more. When the hCG level is below 1000 mIU/mL, only one coloured band appears in the control window but no coloured band appears in the test window. When the hCG level is above 1000 mIU/mL two coloured bands appear, one in each window.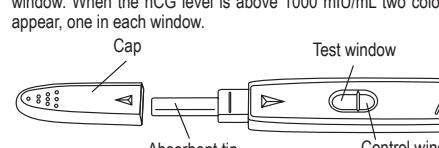

PRECAUTIONS

1. This test is exclusively intended for external use only (in vitro diagnostic use). Do not swallow.

2. Carefully read the instructions before carrying out the test. The test is not reliable unless the instructions are strictly followed.

3. Store between +4°C to +30°C. Do not freeze.

4. Do not use after the expiry date. The expiry date is printed on each of the aluminium pouches and is also printed on the external package. Do not use if the aluminium pouch has been damaged.

5. Do not reuse checkToP®.

6. Keep out of reach of children.

7. After use please dispose of all kit components into domestic waste.


PROCEDURE

The test must be performed after Termination of Pregnancy at a time indicated by your gynaecologist, usually between 10 to 20 days after the first drug intake in Medical ToP

- miscarriage

1. Remove the checkToP® test device from the aluminium foil pouch. Discard the small sachet.

2. Remove the cap and either

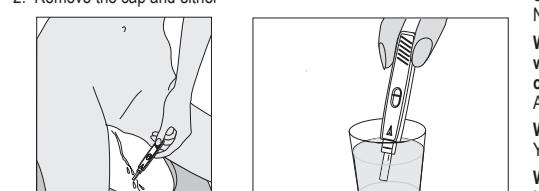- hold the absorbent tip directly in your urine stream for more than 10 seconds. Will I still obtain a correct result?

I left the absorbent tip in my urine for less than 5 seconds. Will I still obtain a correct result?

What if the line in the test window is darker than the line in the control window or the line in the test window is lighter than the line in the control window?

As long as you see a line in each window, the test result is positive.

What should I do when I have the result?

You must follow the advice given by the medical staff managing you.

What if I don't wait the full 5 minutes before reading the test?

If you read the test before 5 minutes, you may not get the test enough time to work, and the test result may not be accurate. DO NOT INTERPRET the results if 10 minutes have elapsed after having brought the absorbent tip into contact with urine.

How accurate is the test?

The checkToP® is very accurate and has been used by health care professionals to confirm the success of Medical Termination of Pregnancy up to 9 WA (hospitals and laboratories).

Evaluation reports show that checkToP® can detect hCG above 1000mIU/mL in over 99% of cases. No test is perfect and even if the test is negative, you should still contact your health care provider if you have symptoms of pregnancy such as bleeding, no period at the expected time, tender breasts, feeling sick, and tummy growing.

Can I use checkToP® to diagnose a pregnancy?

Yes but checkToP® is not able to detect very early pregnancies and only shows positive if the hCG hormone is more than 1000 mIU/mL.

Other pregnancy tests are more sensitive and give a positive result starting from 25 to 50 mIU/mL.

WARNINGS

- checkToP® cannot be used for pregnancies where the initial hCG was lower than 1000mIU/mL.

- Use checkToP® only with a medical treatment ordered by your treating doctor. The clinical interpretation of the test has to be done with according to the medical staff managing you.

Distributed by: NORDIC PHARMA - Under EXELGYN licence
Product advice: info@exelyn.com / +33(0)1.73.00.16.40

Manufactured by: VEDALAB
Rue de l'Expansion, ZAT du Londeau, Cérise B.P.181
61006 ALENCON Cedex France

Note: One band can be darker than the other. The two bands can be any shade of pink-purple and can be lighter or darker than the control. However, you should see two clear parallel bands.

Negative test result:

If only one pink coloured band appears in the control window (the smaller window) and no band appears in the test window (the largest window), the test is negative. Termination of Pregnancy or miscarriage is confirmed.

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